



X-Line™
Radiotherapy Body Contour

50 feet

INDICATIONS

When applied to the patient, the radiopaque lines highlight the body contour in a CT cross-section; the parallel lines appear as dots (see the dots 'D' in Figure 2 on the other side of the box).

X-Line can allow for better visualization of the body contour when it falls within the extended field of view (eFOV). For example, the body contour of the abdominal/pelvic sides of obese patients is commonly distorted or obscured by artifact in the eFOV.

PRECAUTIONS

1. Adhere tightly to the skin; if needed, shaving long body hair beforehand.
2. Care must be used when removing the device from the patient, especially in the elderly, skin adhesive removal solutions may be useful.
3. Do not reuse after removing from the skin.
4. Do not apply on compromised skin.



Target Tape Inc.
201 - 13737 96th Ave
Surrey BC, V3V 0C6
Canada
Target-Tape.com



Non-Sterile



Latex-Free



Do Not Reuse