

elequil aromatabs®

aromatherapy designed for the clinical setting

Instructions for Use

1. Determine the scent level of the Elequil aromatabs® tab by selecting:

MAX SCENT for the full effect of the essential oil or
MIN SCENT for a lighter effect of the essential oil



2. Using your thumb, press down and hold the self-adhesive aromatherapy tab in place on the paper backing. With your other hand, pull and tear the clear cellophane pouch away from you at the preferred NOTCH to expose the scented tab.



3. Lift the self-adhesive aromatabs® off the paper backing.



4. Place a self-adhesive aromatabs on the upper chest or back of clothing or hospital gown during use.



No metal/foil (safe for MRI and/or Intraoperative Procedures)

Video Demonstration

Warnings and Precautions

- The use of peppermint essential oil is contraindicated for patients with cardiac fibrillation or G6PD Deficiency
- The use of peppermint essential oil is not recommended for children under 3 years old
- External use only; do not ingest or swallow
- Do not use if sensitive or allergic to plants as essential oils are natural aromas derived from plants
- Intended for placement on clothing
- To prevent possible skin irritation: do not place directly on skin
- Do not reuse ☒
- Discard after use
- Not made with natural rubber latex
- Non-sterile 