

Instructions for Use:

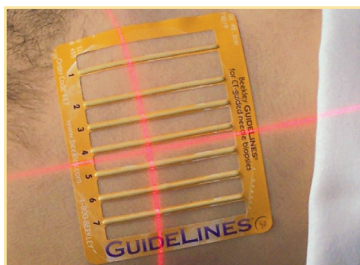


Fig. 1

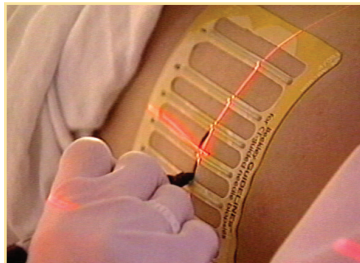


Fig. 2

THE GRID WITH RAISED LINES,
ONLY FROM BEEKLEY MEDICAL

1. Remove backing from GuideLines CT biopsy grid to expose adhesive.
2. Place GuideLines on patient over area of interest.
3. Using the series of lines on the scan, determine preferred area for needle entry. (Fig. 1)
4. Using open windows between the lines, mark needle entry site. If the preferred point of entry falls on a grid line, simply mark on both sides of the line. (Fig. 2)
5. Remove after use.

REF 217 QTY 15 / box
REF 117 QTY 20 / box

 Manufactured by Beekley Corporation
One Prestige Lane, Bristol, CT 06010-7454 USA
Tel: 1.800.233.5539 or +1.860.583.4700 Fax: 1.800.735.1234
beekley.com

BEEKLEY, BEEKLEY MEDICAL, GUIDELINES, THE GRID WITH RAISED LINES, and WHEN YOUR DIAGNOSIS MUST BE RIGHT are Reg. U.S. Pat. & Tm. Off. BEEKLEY is a Registered Community Trademark. BEEKLEY is a registered trademark in Canada.
© 2023 Beekley Corporation. All rights reserved.

WARNINGS AND PRECAUTIONS:

- Single Patient Use Only; Do Not Reuse 
- Not made with natural rubber latex
- Lead-Free
- Non-Sterile 

If any serious incident occurs in relation to this device, report it to Beekley Medical and to the competent authority of the Member State in which the user and/or patient is established.

  
EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands



REV: GL_IFU-WEB_0523