

mark-through GUIDELINES[®]

CT grid for localizing needle entry sites

Beekley
Medical[®]
when your diagnosis
must be right[™]



Fig. 1



Fig. 2



Fig. 3

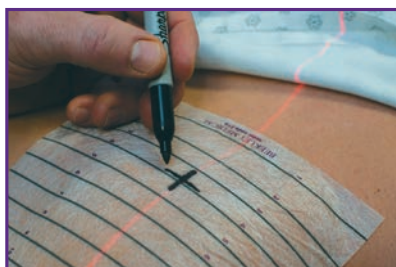


Fig. 4



Fig. 5

Instructions for Use:

1. **Remove grid from backing to expose adhesive.** (Fig. 1)
2. **Place grid on patient directly over area of interest.** (Fig. 2)
3. **Smooth grid onto patient's skin to ensure complete adhesion and eliminate any air bubbles that may occur.**
4. **After scanning, use the series of 11 dots on the image to determine needle entry site.** (Fig. 3)
5. **With a fine point Sharpie[®] marker, make mark where laser crosses the appropriate line on the grid. Ink instantly transfers through the grid material to mark entry site on skin.** (Fig. 4)
6. **Remove grid from patient and discard.** (Fig. 5)





Manufactured by Beekley Corporation
One Prestige Lane, Bristol, CT 06010 USA
Tel: 1.800.233.5539 or +1.860.583.4700 Fax: 1.800.735.1234 beekley.com

BEEKLEY, BEEKLEY MEDICAL, GUIDELINES, MARK THROUGH GUIDELINES, and WHEN YOUR DIAGNOSIS MUST BE RIGHT are Reg. U.S. Pat. & Tm. Off. BEEKLEY is a Registered Community Trademark. BEEKLEY is a registered trademark in Canada. © 2021 Beekley Corporation. All rights reserved.

Sharpie[®] is a registered trademark of Newell Rubbermaid Corporation.

REF 219

WARNINGS AND PRECAUTIONS:

- Non-Sterile 
- Single Patient Use Only; Do Not Reuse 
- Hypoallergenic
- Not made with natural rubber latex
- Lead-Free

If any serious incident occurs in relation to this device, report it to Beekley Medical and to the competent authority of the Member State in which the user and/or patient is established.



EMERGO EUROPE
Prinsesgracht 20
2514 AP The Hague
The Netherlands



REV: MTGL_IFU_0421