

Instructions for use:



1. Remove Bella Blankets® protective coverlet from liner.



2a. Place adhesive side down and smooth by starting at the front of receptor plate while holding the top.



2b. Or apply by starting at the back of receptor plate then pull taut to the front.



3. Use the palm of your hand to smooth and adhere center to top of the receptor plate.



4. From the center, smooth outward to adhere. Remove air bubbles or wrinkles for best image quality.



5. After exam, remove by pulling the Pik-UP™ tab from the corner and discard.

Visit beekley.com for a video demonstration

Practice adhering Bella Blankets protective coverlets a few times to become familiar with application prior to use with patients.

REF 310 22.9cm x 27.9cm

Intended for use on 18cm x 24cm receptor plate

REF 311 29.2cm x 33.0cm

Intended for use on 24cm x 30cm receptor plate

Manufactured by Beekley Corporation
One Prestige Lane, Bristol, CT 06010-7454 USA
Tel: 1.800.233.5539 or +1.860.583.4700
Fax: 1.800.735.1234 beekley.com

BEEKLEY, BEEKLEY MEDICAL, BELLA BLANKETS, and WHEN YOUR DIAGNOSIS MUST BE RIGHT are Reg. U.S. Pat. & Tm. Off. BEEKLEY is a Registered Community Trademark. BEEKLEY is a registered trademark in Canada. Pik-UP is an unregistered trademark of Beekley Corporation. © 2023 Beekley Corporation. All rights reserved.

WARNINGS AND PRECAUTIONS:

- Non-Sterile
- Single Patient Use Only; Do Not Reuse
- Smooth out any air bubbles or wrinkles that may occur as they may show on the image

Bella Blankets are hypoallergenic and are not made with natural rubber latex.

If any serious incident occurs in relation to this device, report it to Beekley Medical and to the competent authority of the Member State in which the user and/or patient is established.

CE

EC REP

EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

MD

REV: BEL_IFU_0423