


AccuGrid[®]

Disposable Specimen Radiography System with Localizing Grid

Beekley
Medical[®]
when your diagnosis
must be right[™]

Instructions for Use

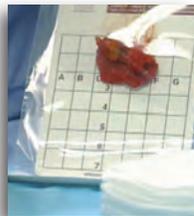

SURGERY:

1. Fill in patient information on designated section at bottom of bag.
2. Place specimen inside bag, centering it on top of grid.
3. Seal bag and transport to Radiology for imaging.



RADIOLOGY:

1. DO NOT OPEN - keep specimen inside bag.
2. Center specimen on grid by manipulating through the bag.
3. Place AccuGrid on mammography unit, apply light compression and take radiograph.
4. Transport specimen to Pathology with copy of specimen radiograph.



PATHOLOGY:

1. Note recorded grid coordinates of specimen.
2. Cut open and remove bag.
3. Use specimen radiograph and grid coordinates to identify area of concern.
4. Ink margins and section right on grid if desired.



Manufactured by Beekley Corporation
One Prestige Lane, Bristol, CT 06010-7454 USA
Tel: 1.800.233.5539 or +1.860.583.4700 Fax: 1.800.735.1234 beekley.com

ACCUGRID, BEEKLEY, BEEKLEY MEDICAL, and WHEN YOUR DIAGNOSIS MUST BE RIGHT are Reg. U.S. Pat. & Tm. Off. ACCUGRID and BEEKLEY are Registered Community Trademarks. BEEKLEY is a registered trademark in Canada.

© 2023 Beekley Corporation. All rights reserved.

WARNINGS AND PRECAUTIONS:

- Non-Sterile 
- Do Not Reuse 
- Observe Biohazard Procedures, Dispose of Properly
- Open or puncture only in biohazard safe environment

If any serious incident occurs in relation to this device, report it to Beekley Medical and to the competent authority of the Member State in which the user and/or patient is established.



EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands



REV: AC_IFU_0423