

# ColdSPOT<sup>®</sup>

Disposable, Self-Adhesive Cold Therapy Packs

Beekley  
Medical<sup>®</sup>

when your diagnosis  
must be right<sup>®</sup>

## Instructions For Use:

1. Pre-freeze ColdSPOT for a minimum of 2 hours before use
2. Remove ColdSPOT and sterile gauze from outer packaging
3. Apply gauze to biopsy or aspiration site
4. Remove adhesive side tab covers (if desired)
5. Center ColdSPOT over site, apply to skin
6. ColdSPOT can be reused by simply refreezing for another 2 hours

ColdSPOT  500, 502, 505

 Manufactured by Beekley Corporation

One Prestige Lane, Bristol, CT 06010-7454 USA

Tel: 1.800.233.5539 or +1.860.583.4700

Fax: 1.800.735.1234 beekley.com Made in the USA

BEEKLEY, BEEKLEY MEDICAL, COLDSPOT, and WHEN YOUR DIAGNOSIS MUST BE RIGHT are Reg. U.S. Pat. & Tm. Off.

BEEKLEY is a Registered Community Trademark.



BEEKLEY is a registered trademark in Canada.

© 2023 Beekley Corporation. All rights reserved.

Samples Only – Not For Resale



### WARNINGS AND PRECAUTIONS:

- Single Patient Use Only; Do Not Reuse 
- Recommended removal after no more than 2 hours to avoid potential adverse skin reaction to adhesive
- Not made with natural rubber latex
- Lead-Free
- Non-Sterile 

If any serious incident occurs in relation to this device, report it to Beekley Medical and to the competent authority of the Member State in which the user and/or patient is established.

Curity All Purpose Sponges REF 8043



 Covidien Ilc, 15 Hampshire Street,  
Mansfield, MA 02048 USA

 Covidien Ireland Limited, IDA Business  
& Technology Park, Tullamore.



REV: CS\_IFU\_0423