

ColdSPOT[®]

Disposable, Self-Adhesive Cold Therapy Packs

Beekley
Medical[®]

when your diagnosis
must be right[®]

Instructions For Use:

1. Pre-freeze ColdSPOT for a minimum of 2 hours before use
2. Remove ColdSPOT and sterile gauze from outer packaging
3. Apply gauze to biopsy or aspiration site
4. Remove adhesive side tab covers (if desired)
5. Center ColdSPOT over site, apply to skin
6. ColdSPOT can be reused by simply refreezing for another 2 hours

ColdSPOT   

 Manufactured by Beekley Corporation

One Prestige Lane, Bristol, CT 06010 USA
Tel: 1.800.233.5539 or +1.860.583.4700
Fax: 1.800.735.1234 beekley.com Made in the USA

BEEKLEY, BEEKLEY MEDICAL, COLDSPOT, and WHEN YOUR DIAGNOSIS MUST BE RIGHT are Reg. U.S. Pat. & Tm. Off. BEEKLEY is a Registered Community Trademark. BEEKLEY is a registered trademark in Canada.

© 2021 Beekley Corporation. All rights reserved.



Samples Only – Not For Resale



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands



WARNINGS AND PRECAUTIONS:

- Single Patient Use Only; Do Not Reuse 
- Recommended removal after no more than 2 hours to avoid potential adverse skin reaction to adhesive
- Not made with natural rubber latex
- Lead-Free
- Non-Sterile 

If any serious incident occurs in relation to this device, report it to Beekley Medical and to the competent authority of the Member State in which the user and/or patient is established.

Curity All Purpose Sponges REF 8043



 Covidien LLC, 15 Hampshire Street, Mansfield, MA 02048 USA
 Covidien Ireland Limited, IDA Business & Technology Park, Tullamore.



REV: CS_IFU_0621