

MARGINMAP[®]

SPECIMEN ORIENTATION CHARMS

Instructions for Use

1. Thread charm onto suture, twist base and charm will easily detach.






2. Suture charm onto the corresponding specimen margin. Use a secure, but not taut, stitch.



3. Repeat until the desired number of margins are indicated on the specimen.
note: all six charms do not need to be used



WARNINGS AND PRECAUTIONS:

- Device and inner package sterile unless outer package has been opened or damaged. 
- No part of this product should be left in the patient for any reason
- Single patient use only; Do not reuse 
- Do not resterilize 
- Not made with natural rubber latex
- Lead-Free