

# Pinnacle®

NEEDLE-LOC NEEDLE PROTECTOR

Beekley  
Medical®  
when your diagnosis  
must be right™



## Instructions for Use

1. Remove Pinnacle from packaging. Press open into formed shape of product. Fold crease on top closure tab and fold up the four securing tabs at the base.
2. Remove adhesive liners from the four securing base tabs.
3. Place Pinnacle over needle-loc needle using side windows and top opening to center. Secure it to the breast with the four adhesive base tabs.
4. Once Pinnacle is secure, fold top closure tab over top of product and insert into slit on side.



REF 330

Manufactured by Beekley Corporation  
One Prestige Lane, Bristol, CT 06010 USA  
Tel: 1.800.233.5539 or +1.860.583.4700 Fax: 1.800.735.1234  
beekley.com

BEEKLEY, BEEKLEY MEDICAL, PINNACLE, and WHEN YOUR DIAGNOSIS MUST BE RIGHT are Reg. U.S. Pat. & Tm. Off. BEEKLEY is a Registered Community Trademark. BEEKLEY is a registered trademark in Canada.

© 2021 Beekley Corporation. All rights reserved.

#### WARNINGS AND PRECAUTIONS:

- Single Patient Use Only; Do Not Reuse 
- Hypoallergenic
- Not made with natural rubber latex
- Non-Sterile 

If any serious incident occurs in relation to this device, report it to Beekley Medical and to the competent authority of the Member State in which the user and/or patient is established.



EMERGO EUROPE  
Prinsesgracht 20  
2514 AP The Hague  
The Netherlands



REV: PIN\_IFU\_0421