# Our Imaging Center Offers an Added Layer of Protection During Your Mammogram


Bella Blankets® protective coverlets will be used to provide you with an **extra** level of care

*Bella Blankets*®  
Protective Coverlets for Mammography

Our imaging center uses Bella Blankets<sup>®</sup> protective coverlets to provide you the best in patient care

### What are Bella Blankets?

Bella Blankets are protective coverlets that attach to the mammography equipment.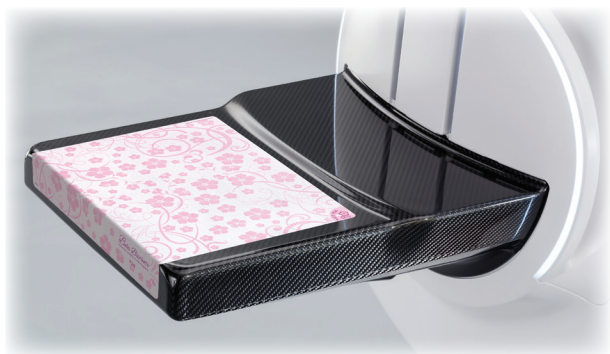

### Are Bella Blankets sanitary?

Although the technologists sanitize the equipment between patient appointments, Bella Blankets are an added level of protection and help to act as an **additional sanitary barrier**. They are used for your exam only and then discarded afterwards.

## How will Bella Blankets feel during my mammogram?

It will feel less cold to the touch because Bella Blankets remove the chill from the equipment.

## What clinical impact does Bella Blankets offer?

Our top priority is to get the best image possible. Bella Blankets have a unique material that helps hold the breast in place, especially if you are perspiring. Bella Blankets may help the technologist get **more breast tissue**<sup>1</sup> on your image for the radiologist to review.

## Why are Bella Blankets so thin?

Bella Blankets are thin by design and may help the technologist get **more breast tissue**<sup>1</sup> because of the thin physical barrier between you and the equipment.

## Are Bella Blankets suitable for sensitive skin?

Bella Blankets are hypoallergenic and are not made with natural rubber latex.

## What else can Bella Blankets do to make my experience more pleasant?

Some women have cuts under their breast or infections and it may be painful. Bella Blankets can help decrease the discomfort of cuts which could be felt during your mammogram.

---

<sup>1</sup> Wagner, S. and A Merkulov, MD. 2013. "Comparison Study: Impact of Thin Protective Coverlets on Positioning and Tissue Acquisition in Breast Imaging."

## **Why Bella Blankets?**

Your experience is important to us! We want to make your mammogram as pleasant as possible and provide you with the peace of mind that we take your breast health seriously.

## **Will Bella Blankets cost me extra money?**

No! Bella Blankets are an added level of service at no additional charge!

## **Are Bella Blankets safe?**

Yes, the FDA (U.S. Food & Drug Administration) cleared Bella Blankets in March 2008.

---

*We care about your health*

*We care about your well-being*

*We care about you*


---

***We know you have  
choices, thank you for  
selecting us for your  
mammogram today!***


## Your Breast Health is Important to Us!

This is a reminder to perform a regular Breast Self-Exam.

According to the American Breast Cancer Foundation (ABCF), there are 3 recommended ways to perform a self-breast exam: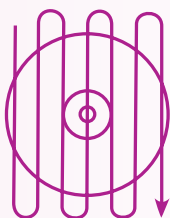

vertical

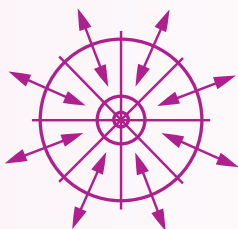wedge

circular

If you choose to do a breast self-exam (BSE), you should have your technique reviewed by a medical professional.

Visit [abcf.org](http://abcf.org) for more information.

WARNINGS AND PRECAUTIONS:

- Non-Sterile 
- Single Patient Use Only; Do Not Reuse 
- Smooth out any air bubbles or wrinkles that may occur as they may show on the image

See package insert for Instructions for Use and other safety related information or visit [beekley.com/ifu](http://beekley.com/ifu)

Bella Blankets are hypoallergenic and are not made with natural rubber latex.

 Manufactured by Beekley Corporation  
One Prestige Lane, Bristol, CT 06010 USA  
Tel: 1.800.233.5539 or +1.860.583.4700  
Fax: 1.800.735.1234 [beekley.com](http://beekley.com)

BEEKLEY, BEEKLEY MEDICAL, BELLA BLANKETS, and WHEN YOUR DIAGNOSIS MUST BE RIGHT are Reg. U.S. Pat. & Tm. Off. BEEKLEY is a Registered Community Trademark. BEEKLEY is a registered trademark in Canada. PIK-UP is an unregistered trademark of Beekley Corporation.

© 2021 Beekley Corporation. All rights reserved.

**Beekley  
Medical**<sup>®</sup>  
when your diagnosis  
must be right<sup>®</sup>

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Elaborado por Beekley Corporation
© 2021 Beekley Corporation. Todos los derechos reservados.