

CaraClip™


wire protector


Sterile, adhesive clips to secure and hold guidewires, catheters, sheaths/tubing or other accessories during interventional, surgical, and other procedures.

Instructions For Use:

1. Remove liner backing to expose adhesive.
2. Apply firmly to ensure adhesion.
3. To insert or remove device(s) into/from lumen of CaraClip:
 - a. press down on the long arm of the clip, or
 - b. advance through lumen

Note: NOT FOR PINNING

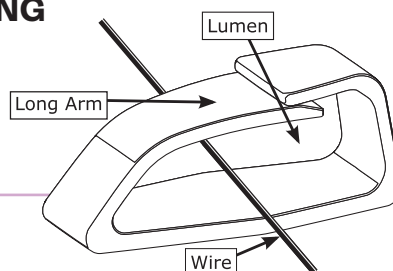




 Manufactured by Beekley Corporation
One Prestige Lane, Bristol, CT 06010-7454 USA
1.800.233.5539 or +1.860.583.4700 beekley.com

CARA CLIP is an unregistered trademark of Beekley Corporation.
® – Reg. U.S. Pat. & Tm. Off.
© 2023 Beekley Corporation. All rights reserved.
Pat. – beekley.com/patents

STERILE R

REF 431

WARNINGS AND PRECAUTIONS:

- Do not use if product sterile packaging is compromised. 
- Single patient use only; Do not reuse. 
- Do not resterilize. 
- Does not contain natural rubber latex.

**Beekley
Medical®**
when your diagnosis
must be right®

REV: CC_IFU_1023