## iFIX INSTRUCTION FOR USE

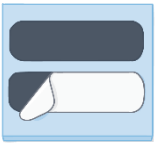
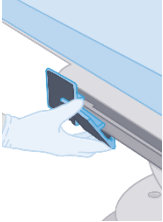
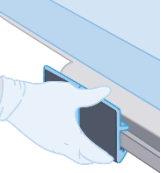
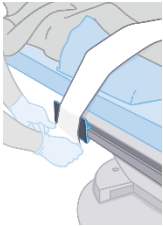
	<b>iFIX Products</b> .....	3
	<b>iFIX Products Illustrations</b> .....	5
<b>EN</b>	<b>iFIX Product Line Instructions for Use</b> .....	7
<b>DE</b>	<b>iFIX Produkt Linie Gebrauchsanweisung</b> .....	10
<b>DA</b>	<b>Brugsanvisning for iFIX-produktlinjen</b> .....	13
<b>ES</b>	<b>Instrucciones para el uso de la línea de productos iFIX</b> .....	16
<b>FR</b>	<b>Mode d'emploi pour la gamme de produits iFIX</b> .....	19
<b>IT</b>	<b>Istruzioni per l'uso della linea di prodotti iFIX</b> .....	22
<b>NL</b>	<b>iFIX Productlijn Gebruiksaanwijzing</b> .....	25
<b>PL</b>	<b>Linia produktów iFIX Instrukcja użycia</b> .....	28
<b>PT</b>	<b>Instruções de Utilização da Linha de Produtos iFIX</b> .....	31
<b>SV</b>	<b>Bruksanvisningar för produktserien iFIX</b> .....	34

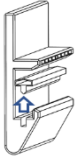
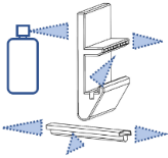
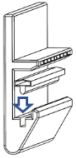
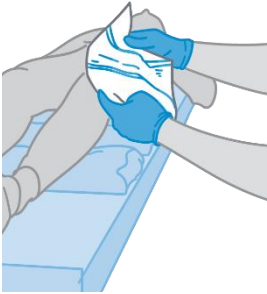
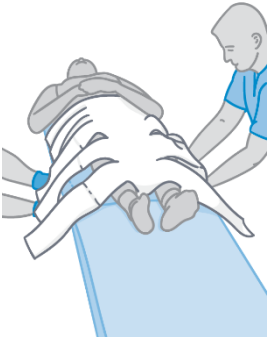
## iFIX Products

REF	Product	
<b>iFIX Fleece</b>		
31618	iFIX Fleece 5	QTY: 1; 5 cm x 30 m
16304	iFIX Fleece 15	QTY: 1; 15 cm x 200 m
17674	iFIX Fleece 50	QTY: 1; 50 cm x 30 m
<b>iFIX Spider Fleece</b>		
40286	iFIX Spider Fleece Arm M	QTY: 1; 50 x 45 cm
40287	iFIX Spider Fleece Leg M	QTY: 1; 75 x 75 cm
40288	iFIX Spider Fleece Body S	QTY: 1; 100 x 90 cm
40289	iFIX Spider Fleece Body M	QTY: 1; 125 x 90 cm
40290	iFIX Spider Fleece Body L	QTY: 1; 150 x 90 cm
40291	iFIX Spider Fleece Body XL	QTY: 1; 175 x 90 cm
<b>iFIX Head Mask</b>		
26102	iFIX Head Mask XS	QTY: 10; 33 x 41 cm
26105	iFIX Head Mask S	QTY: 10; 33 x 43 cm
26104	iFIX Head Mask M	QTY: 10; 33 x 45 cm
26103	iFIX Head Mask L	QTY: 10; 33 x 47 cm
29341	iFIX Head Mask XL	QTY: 10; 33 x 49 cm
35080	iFIX Head Mask L <i>elongated</i>	QTY: 10; 58 x 62 cm

REF	Product	
<b>iFIX Patch</b>		
16180L001	iFIX Patch 15	QTY: 8; 15 x 10 cm
16180L002	iFIX Patch 25	QTY: 6; 25 x 10 cm
16180L005	iFIX Patch 50	QTY: 4; 50 x 10 cm
16180L007	iFIX Patch 75	QTY: 4; 75 x 10 cm
32982	iFIX Patch 15x6	QTY: 3; 15 x 6 cm
<b>iFIX Patch Adapter</b>		
16179L001	iFIX Patch Adapter 50	QTY: 4; 50 x 15 cm
<b>iFIX Slot Adapter</b>		
38037	iFIX Slot Adapter 2.0/30	QTY: 4; 30 x 12 cm, Ø 3.0 mm
17622L003	iFIX Slot Adapter 3.0/30	QTY: 4; 30 x 12 cm, Ø 4.0 mm
16192L003	iFIX Slot Adapter 8.0/30	QTY: 4; 30 x 12 cm, Ø 9.5 mm
33224	iFIX Slot Adapter 10.0/30	QTY: 4; 30 x 12 cm, Ø 11.5 mm
35193	iFIX Slot Adapter Trapezoid 12.0/9	QTY: 8; 9 cm x 14 cm x 13.3 mm
39775	iFIX Slot Adapter Wing 3.0/30	QTY: 4; 30 x 13,5 cm, Ø 4.0 mm, wing-shaped rod
<b>iFIX Side Rail Adapter</b>		
42642-EU	iFIX Side Rail Adapter	QTY: 4; 12,5 x 8 x 2 cm
44824	iFIX Side Rail Adapter Spare Patch	QTY: 10; 12 x 10,5 cm

## iFIX Products Illustrations

A1	B1	C1	D1
			
D2	D3	E1	E2
			
F1	F2	F3	
			

<p>G1</p> 	<p>G2</p> 	<p>G3</p> 
<p>H1</p> 	<p>H2</p> 	
<p>H3</p> 	<p>H4</p> 	

EN

## iFIX Product Line Instructions for Use

### Intended Use

The iFIX Product Line is a non-sterile product, intended for patient stabilization over a short period of time, during invasive and non-invasive procedures, by securing the position of the patient's anatomy with intact skin outside the sterile field.

### Description

iFIX is a patient stabilization system, indicated for medical procedures during which the patient is required to maintain a specific position over a short period of time. The stretchy fleece is intended for single patient use and, as a single use component, lowers the risk of cross-contamination. It contours to the patient's anatomy and secures them in the desired position through its connection to the reusable iFIX anchor products. Depending on the table design, an iFIX Patch that adheres to the table surface, an iFIX Slot Adapter that slides into the channels or an iFIX Patch Adapter which connects to standard hook fasteners can be chosen. The micro anchors of the iFIX Patches and Adapters strongly secure the iFIX Fleece and iFIX Spider Fleece to ensure optimal patient stabilization and minimizing motion artifacts.

### Warnings and Precautions

**Warning:** The system may only be used by medical professionals.

**Warning:** iFIX Fleece, iFIX Spider Fleece and iFIX Head Mask are products for single patient use only. Do not reuse iFIX Fleece, iFIX Spider Fleece and iFIX Head Masks.

### Selection of the anchor product

#### iFIX Slot Adapter

*In case the table contains channels:*

1. Select an appropriate iFIX Slot Adapter size and shape.
2. Slide the iFIX Slot Adapters to the area of the patient's anatomy to be stabilized (See **A1**, p. 5). While positioning, make sure that no collisions with the existing equipment take place during use.
3. Use additional iFIX Slot Adapters if required.

#### iFIX Side Rail Adapter (SRA)

*In case the table is equipped with a side rail:*

1. Hook in the SRA on the bottom edge of the rail first (See **F1**, p. 5).
2. Push the SRA towards the table till it clicks on the rail and the patch surface becomes parallel to the rail's surface (See **F2**, p. 5).
3. Attach the iFIX Fleece using its textured side to the SRA (See **F3**, p. 5).

#### iFIX Patch

*In case there is a flat surface available*

1. Remove the liner from the iFIX Patch.
2. Adhere the iFIX Patches to the lateral sides of the patient support device, table or another desired surface (See **B1**, p. 5).
3. Expose the anchor side of the iFIX Patch by removing the white cover (See **C1**, p. 5).
4. Adhere additional iFIX Patches if required.

#### iFIX Patch Adapter

*In case the table contains a standard hook fastener:*

1. Attach the loop side of the iFIX Patch Adapter to the existing hook fastener on both sides of the patient table.

2. Expose the anchor side of the iFIX Patch Adapter by removing the white cover (See **C1**, p. 5).
3. Add additional iFIX Patch Adapters and/or cut iFIX Patch Adapters if required.

#### Patient stabilization

##### iFIX Fleece and iFIX Spider Fleece

1. Estimate the required length of iFIX Fleece and cut or select an appropriate iFIX Spider Fleece size for the patient, place it in the middle of the patient and unfold the iFIX Spider Fleece. (See **H1-H2**, p. 6).
2. Attach the iFIX Fleece / iFIX Spider Fleece using its textured side to an anchor product on the opposite side of the table (See **D1**, p. 5 and **H3**, p. 6).

**Warning:** iFIX Fleece has a plain side facing upwards when pulled out of the fleece dispenser, and a textured one on the opposite side. Only the latter can be used for the connection to the anchor products. Using the wrong surface of the fleece may lead to patient injury.

3. Pull the iFIX Fleece / iFIX Spider Fleece tightly, up and over the patient (See **D2**, p. 5 and **H3**, p. 6).

**Warning:** Do not apply the iFIX Fleece / iFIX Spider Fleece too tight, or it can restrict the blood circulation and the patient's breathing.

4. Adhere the iFIX Fleece / iFIX Spider Fleece to the anchor product on the other side of the table. If necessary, adjust the iFIX Fleece / Spider Fleece to increase or decrease stabilization level. (See **D3**, p. 5 and **H4**, p. 6).

**Warning:** Confirm the stability of the setup before conducting the procedure and readjust if necessary. Forgetting it may lead to patient injury.

5. If necessary, repeat this procedure with additional iFIX Fleece layers to increase stabilization.

##### iFIX Head Mask

1. Select an appropriate iFIX Head Mask size for the patient.

**Warning:** Using the wrong Head Mask size may lead to patient injury.

2. Place the mask over the patient's face ensuring eyes, nose, and mouth are exposed (See **E1**, p. 5).

**Warning:** Do not apply the iFIX Head Mask too tight, or it can restrict the blood circulation and the patient's breathing.

3. Attach the iFIX Head Mask flaps to an anchor product. **Warning:** Confirm the stability of the setup before conducting the procedure and readjust if necessary. Forgetting it may lead to patient injury.
4. If necessary, adjust iFIX Head Mask to increase or decrease stabilization (See **E2**, p. 5).

##### Cleaning, refurbishing and disposal

1. After use, dispose of the iFIX Fleece / iFIX Spider Fleece or the iFIX Head Mask.

**Warning:** Reusing an iFIX Fleece / iFIX Spider Fleece or an iFIX Head Mask may lead to cross-infection.

2. Clean all reusable anchor products after use. Residues of iFIX Fleece / iFIX Spider Fleece can be removed using commercially available lint roller.
3. Apply a disinfection agent to the reusable products. Make sure there is complete wetting of the anchor surfaces with disinfection agent. Recommended agents: alcoholic (e. g. Icidin Liquid or Mikrozyd AF Liquid) or quaternary ammonium (e.g. Meliseptol) surface disinfectants. When disinfecting the Side Rail Adapter: take care to disassemble the adapter prior to disinfection (See **G1**, p. 6) and only use alcoholic surface disinfectants as a spray in order to treat adequately all surfaces, holes, lumen and edges (See **G2**, p.6). Allow agent to take effect during at least 5 minutes before reassembling the anchor (See **G3**, p. 6).

4. In case iFIX Fleece does not attach to the Side Rail Adapter anymore, remove the Patch and replace it by a new Side Rail Adapter Spare Patch.

#### Product Information

- iFIX is not made with natural rubber latex.
- iFIX is hypoallergenic.
- iFIX is safe for use in MRI environment.

#### Symbol definitions

The device is safe for use in MRI Environment

The device is not safe for use in MRI Environment

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© iSYS Medizintechnik GmbH
Interventional Systems is a registered trademark
iSYS is a registered trademark
iFIX is a registered trademark
iSYS Medizintechnik GmbH